

Electrocardiogram (ECG) Confirmation in MUSE for Providers

Cerner PowerChart Ambulatory EDUCATION

ECG Confirmation in MUSE

1. Open MUSE from Citrix and log in.



- 2. Click on ECG's to Read.
- 3. Double click on the patient name to open the ECG.



- 4. Review the interpretation and edit as needed.
- 5. Click Confirm.

System	Current ECG	Serial Presentation	Measurements	First Previous ECG	Oldest ECG	
2 2	Patient ID:	004000058	Vent	icular, atrial rate: 97	94 bpm	
	Name:	ZZZZVISIONTEST	, PATIENT	PR, QRS:	70 ms	Atrial fibrillation
Go	Test Date/Time:	05-Dec-2019 13:50:17		QT, QTc: 312	396 ms	Abnormal ECG
	Sex:	Female 💌 Race:	Caucasian	PRT Axis:	71 62 °	
	Ht, Wt (cm, kg):	Age/DOB:	01-Jan-1932 (87	Years) BP:	/ mm Hg	
Confirm	Room Number:	I	Referring Phys:			4
0.6 5	Cart Number:	22	Ordering Phys:	TEST, PHYSIC	IAN	
	Option Number:		Acquiring Tech:	BM		
Waveform Tools	Order Number:	2770311165	Editor:			
	Visit Number:	A0054527485	Fellow:			
	Secondary ID:	01342857	Confirmed by:			
	:		Test Reason:			Edit Date/Time: 15-Dec-2021 15:14:21
	SURG.DA	ATE:	CC	DMMENTS:		
Print			7			
			A . A			
50						a a la construction de la constr
					ave .	••••

- The order has been completed and the charge has been submitted.
- The ECG is now available for viewing in PowerChart.

Workflow Page

Diagn

Results Review

iagnostics	Results Review	Cardiology Procedures/Studies	Resting ECG	ECG Image
		12/5/2019 13:50 EST	Resting ECG	ECG Image
ECG Image				