


## Spirometry Ordering

1. Place the Perform Spirometry order in PowerChart.

**Note:** This image depicts the Perform Spirometry order being placed from the Ambulatory QOC- Primary Care Workflow.

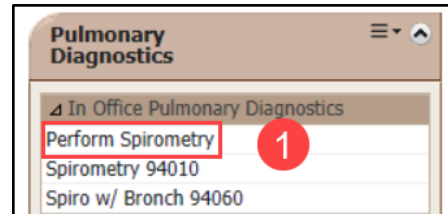

## Spirometry Reading

After the clinician performs the Spirometry exam:



1. Log in to Midmark IQmanager on a clinic desktop.




2. Click on Patient List.



3. Find the correct patient and double click the name.
4. Click View Report.

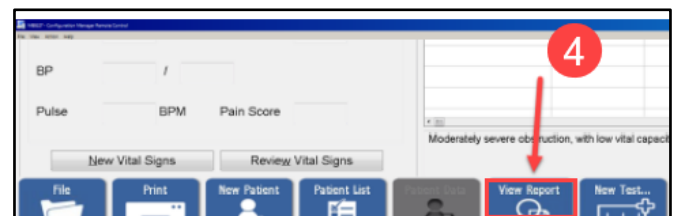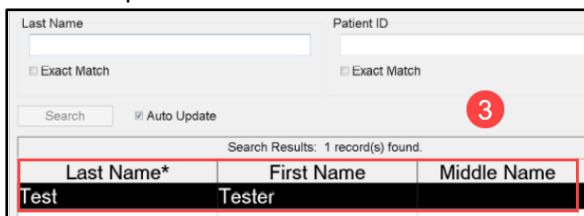

5. Enter in the user's name in the Reviewed By field and then type in the Interpretation. Once complete, click Exit.

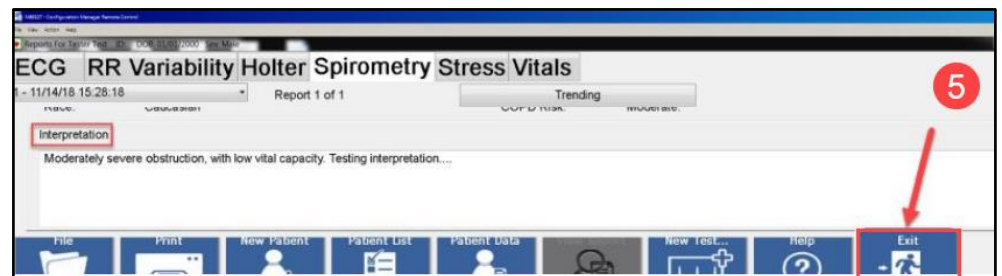

6. Click Yes to save the report.
7. Log in to PowerChart, open the patient's chart, navigate to quick orders, and order the correct spirometry charge.

