

# IntelConnect Access to View Radiology Images for Providers and Clinical Staff

Cerner PowerChart, FirstNet, Ambulatory **EDUCATION**

## Using IntelConnect

The IntelConnect application offers a comprehensive image viewing platform preferable over the application embedded into the patient's chart, with popular features including printing and multiple image view.


Additionally, the IntelConnect application can be used to access radiology images completed at any of the MHC facilities:

- MHC Cadillac Hospital
- MHC Munson Medical Center
- MHC Manistee Hospital
- MHC Charlevoix Hospital
- MHC Paul Oliver Memorial Hospital
- MHC Otsego Memorial Hospital
- MHC Grayling Hospital

## Accessing IntelConnect

Within **PowerChart or FirstNet**:


1. Click on the Resources icon in the top toolbar.
2. The Clinical Resources page will open. Select Intelconnect.

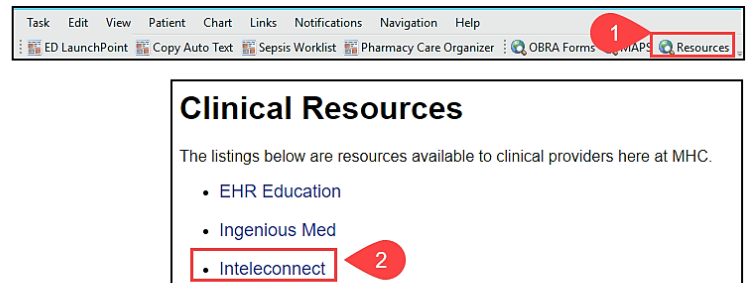

**OR**



From the **Intranet** navigate to a Munson home page:

1. Select My Apps.
2. Select Intelconnect.

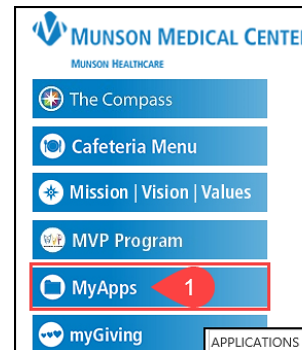

After selecting IntelConnect:

3. Log into IntelConnect using your assigned system login and password.
4. Enter the patient's name or MRN.
5. Select the drop down to add DOB.

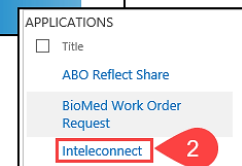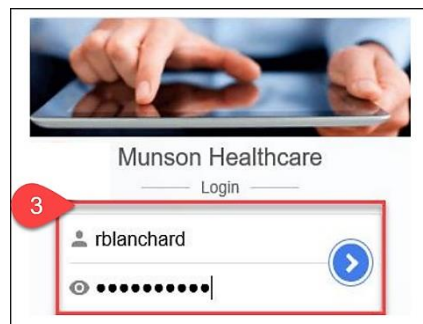

For off-site access, complete authentication using Google Authenticator (also used for ePrescribe).



# IntelConnect Access to View Radiology Images for Providers and Clinical Staff

Cerner PowerChart, FirstNet, Ambulatory EDUCATION

## Viewing Images and Reports

1. Select an exam.

Exam Description	Exam Date ▼	Radiologist	Order Status	Organization	Image Availability
US RENAL	2021-Mar-03 02:34 PM	Testerman, Shannon	✓ Final Report (ZZ)	MMC	NO IMAGES
US RENAL/BLADDER	2021-Mar-03 02:34 PM	Testerman, Shannon	✓ Final Report (ZZ)	MMC	NO IMAGES
CR CHEST 2 V	2021-Feb-26 10:47 AM	Testerman, Shannon	✓ Final Report (ZZ)	MMC	ONLINE

2. View images and reports in the exam window.

**Zzzradiologyuseonly, Do Not Use**  
 DOB: 1976-Nov-02 (46 Y) Gender: Female Alternate ID: 214652 (MRN)

View PDF... Launch IntelViewer

**DX WRIST COMP MIN 3 V BILAT**  
 Priority: **A Routine**  
 Accession Number: 00000GD210022451  
 Referring Physician: NO PHYSICIAN  
 Organization: MMC  
 Status: ✓ Final Report (ZZ)  
 Study Date: 2021-Feb-23 08:35 AM  
 Radiologist: Damiano, Sarah  
 Location: MMC RAD MAIN

Final Report (ZZ)

EXAM:  
WRIST COMP MIN 3 V BILAT

Ordering Provider: PHYSICIAN, NO

HISTORY:  
test

Electronically Signed By:  
Damiano, Sarah R.T.(R)

CR X WRIST LAT LEFT (1 image)    CR X WRIST NAVICULAR... (1 image)    CR X WRIST LAT LEFT (1 image)

Click the image to open the viewing window, and to access the viewing tools.

3. With the exam window open, references including Toolbar Function information can be accessed via the (?) icon.

Includes Toolbar Function information.

What's New...

Quick Reference

Release Notes

NM RES #1000 UAT Results

About