


SkyVue Viewing Two Images for Providers, Nurses, Clinical Staff

Cerner PowerChart, FirstNet and Ambulatory EDUCATION

Comparing Images from a Different Series

To compare images from a different series (compare today's chest x-ray against yesterday's chest x-ray):

1. Click the **Series icon**, located on the Toolbar. Drag mouse over the number of views desired simultaneously.
2. From the top Menu Bar click **Workspace**.
3. Click **Workspace Gadgets**.
4. Click **History**.

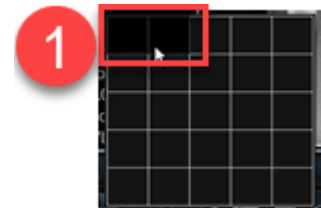

5. Images are selectable from the **History Toolbar**.
 - To adjust the view, hover over blue lines, once black arrows appear, left click and drag to preferred view.




6. Double click the desired image in the History Pane to load the thumbnails into the Thumbnail pane.



SkyVue Viewing Two Images for Providers, Nurses, Clinical Staff

Cerner PowerChart, FirstNet and Ambulatory EDUCATION

7. From the Thumbnail views, **left click and drag** the image into the Viewport.

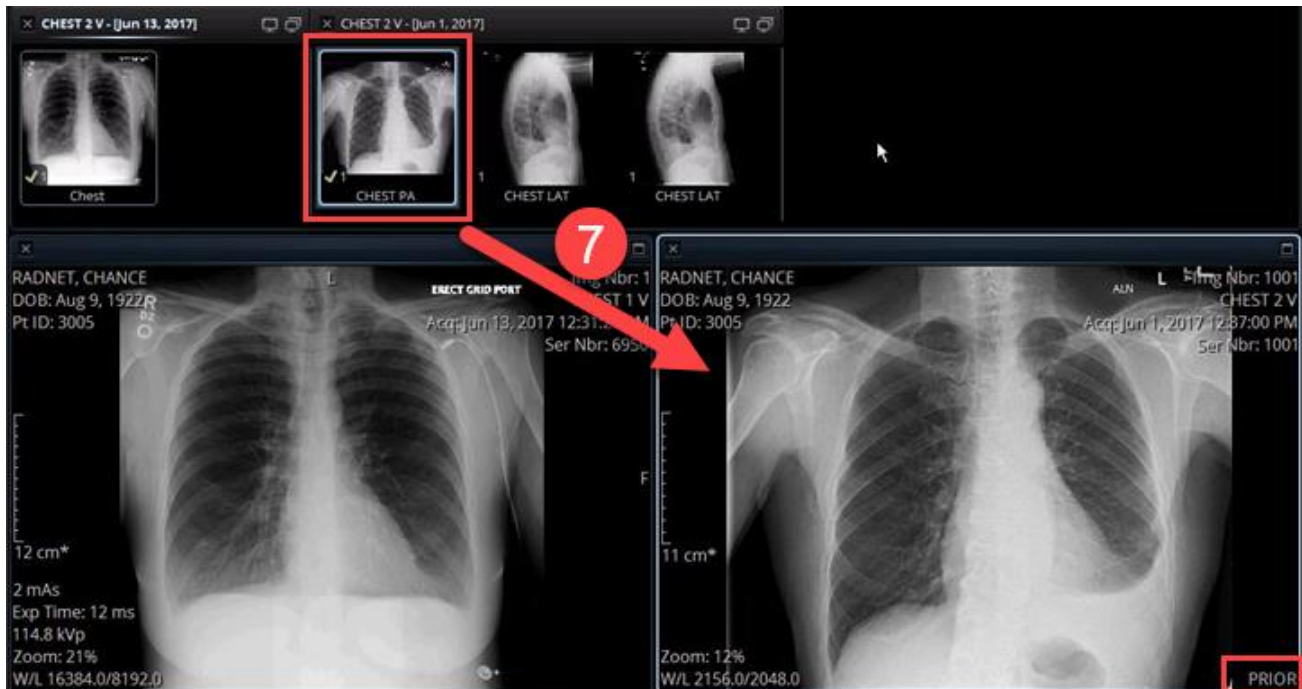

8. To Default the History View

- **Open Histories.**
- Click Workspace. **Click Save Workspace.**
- This is a one-time action to set the default.

