Accessing InteleConnect to View Radiology Images for Providers and Clinical Staff

Using InteleConnect

0

- Use to access radiology images rather than via the patient's chart.
- Use InteleConnect to view images from the following MHC facilities:
 - Cadillac Hospital

o Grayling Hospital

Charlevoix Hospital

- Kalkaska Memorial Health Center
 Munson Medical Center
- Paul Oliver Memorial Hospital
- Charlevoix Hospital
 Manistee Hospital
- Select My Apps on the Intranet (1), click InteleConnect (2) then log into InteleConnect (3).
 - 🔘 Cafeteria Menu Halogen HBI / Insight ✓ Daily Check-In Inteleconnect O CREED Munson Healthcare **KHA EPM Windows Client** 🕐 Employee Logo Gear Login Kronos - Munson 6.3 rblanchard 1 Events Calendar LAB - Tera Term Hission | Vision | Values MVP Program Stay logged into InteleConnect for your shift MyApps to access images quickly.
- Enter the patient's name or MRN. Select the drop-down triangle to add DOB.

InteleConnect.EV

Search patient name, id, accession, or MRN Searching for patients with compound names?

- Accessing InteleConnect from Meditech
- To access InteleConnect from within Meditech:
 - Click global icon bottom right corner on screen.









InteleConnect.

JOB AID

1 of 3



Accessing InteleConnect to View Radiology Images for Providers and Clinical Staff

JOB AID

• **Click** on the External Links.



• Link will open InteleConnect.

Munson Healthcare
± µsemame
Password
Forgot Password?
Apply for an account

Viewing Images

• Click on any exam.

Zzzzradiologyuseonly, Do N DOB: 1976-Nov-02 (43 Y) Gender: Femal	lot Use le Patient ID: 499664				
Exam Description	Exam Date	Radiologist	Order Status	Organization	Image Availability
Interventi NEURO IR PROCEDURE	2020-Aug-07 09:24 AM	Damiano, Sarah	Final Report (ZZ)	MMC	NO IMAGES
US OB COMPLETE	2020-Jul-24 11:31 AM		Order Complete (CM)	MMC	ONLINE
US OB 1ST TRIMESTER	2020-Jul-24 11:30 AM		Order Complete (CM)	MMC	ONLINE
			_		



Accessing InteleConnect to View Radiology Images for Providers and Clinical Staff

JOB AID

• The exam window opens to view images and reports.

View PDF Launo	ch InteleViewer Grant Access Add Impression Edit	
DX CHEST 2 V	A.D. <i>(</i>	144 (146 (146 - 14
Priority	A KOUTINE	Click on the image to
Accession Number		view in full screen
Organization	MMC	
Status	Final Report (ZZ)	
Study Date	2020-Mar-09 02:20 AM	and the second s
Radiologist	Cousins, Doreen	(1 imane)
Location	MMC RAD MAIN	(i inage)
HISTORY: test COMPARISON: None TECHNIQUE:		
FINDINGS: Lungs: Normal. Cardiomediastinals Additional Commer	ι. silhouette: Within normal limits. ιts: None.	
IMPRESSION:		